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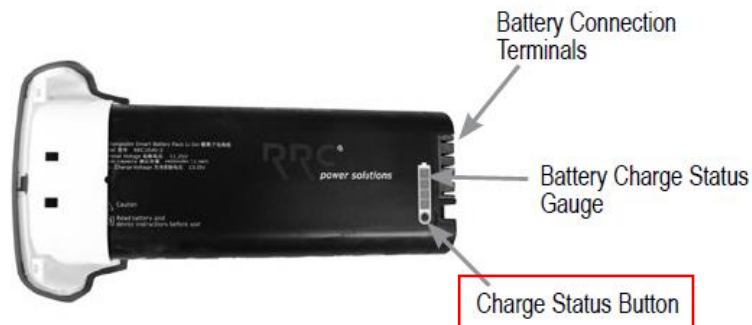
## SYMBOL DEFINITIONS

	Danger - No smoking near patient or device. ⓘ This symbol has a red circle and diagonal bar on the product label.		Power Button		Direct Current (DC Power)		Alarm Silenced
	It is mandatory to read and understand the operating instructions prior to use. ⓘ This symbol has a blue background on the product label.		Increase Prescription Setting Button		Class II electrical protection – double insulated		TUV Rheinland
	Do not use near heat or open flames. ⓘ This symbol has a red circle and diagonal bar on the product label.		Decrease Prescription Setting Button		Type BF applied part		Manufacturer
	Do Not Disassemble ⓘ This symbol has a red circle and diagonal bar on the product label.		Battery Status Indicator Symbol		Catalog Number		European Representative
	Use no Oil, Grease or Lubricants ⓘ This symbol has a red circle and diagonal bar on the product label.		Normal Oxygen (Green)		Serial Number		European Rep CE Mark
	RTCA / DO-160G Section 21 Category M and Section 20 Category T - The manufacturer of this POC has determined this device conforms to all applicable FAA acceptance criteria for POC carriage and use on board aircraft. ⓘ Text shown with this symbol is red on the product label.		Low Oxygen (Yellow)		No MR (MR Unsafe) Unsafe for Magnetic Resonance Environment		
	General Warning ⓘ This symbol is used throughout this manual to indicate hazardous situations to avoid.		Service Required (Red)		Federal (U.S.A.) law restricts this device to sale by or on the order of a physician		
	Important Information ⓘ This symbol is used throughout this manual to indicate important information you should know.		IP22		IP22 Ingress Protection - Protected against finger access to hazardous parts; protected against vertically falling water drops when enclosure is tilted up to 15°		
	Note and Information Symbol ⓘ This symbol is used throughout this manual to indicate notes, useful tips, recommendations and information.				This device contains electrical and/or electronic equipment that must be recycled per EU Directive 2012/19/ EU- Waste Electrical and Electronic Equipment (WEEE)		

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